



CLUNGENE® COVID-19 Antigen Rapid Test

SUMMARY











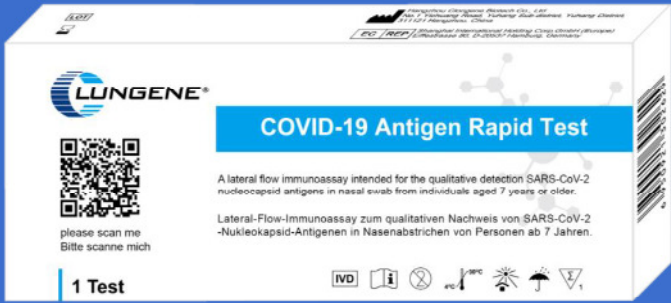

01

For Self-Testing

CLUNGENE® COVID-19 Antigen Rapid Test

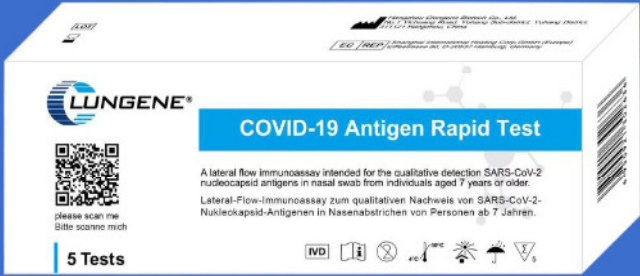

Product		Specifications	MEAS/BOX	
 <p>The image shows a white box for the LUNGENE COVID-19 Antigen Rapid Test. The box features the LUNGENE logo, a QR code, and text in English and German. It specifies '1 Test' and includes various safety and handling icons.</p>		1 Test/Box	165*68*20mm	
Extraction Reagent x1	Test Cassette x1	Tube x1	Swab x1	
				
Packing Carton Information		MEAS/CTN	TESTS/CTN	Estimated G.W./CTN
		52*35*57cm	400tests	14.2kg

Product		Specifications	MEAS/BOX
 <p>The image shows a white box for the LUNGENE COVID-19 Antigen Rapid Test. The box features the LUNGENE logo, a QR code, and text in English and German. It specifies '5 Tests' and includes various safety and handling icons.</p>		5 Tests/Box	190*70*38mm
Extraction Reagent x5	Test Cassette x5	Tube x5	Swab x5
			
Packing Carton Information		MEAS/CTN	Estimated G.W./CTN
		59*40*58cm	20.3kg



Tests zur Eigenanwendung durch Laien

Das BfArM hat die ersten Sonderzulassungen nach §11 Absatz 1 Medizinproduktegesetz (MPG) von Antigen-Tests zur Eigenanwendung durch Laien (Selbsttests) zum Nachweis von SARS-CoV-2 erteilt. Weitere Informationen zur rechtlichen Grundlage und den dabei geprüften Anforderungen finden Sie weiter unten auf dieser Seite unter dem Menüpunkt „Hinweise zur Sonderzulassung von Antigen-Tests durch das BfArM“.

Es handelt sich um folgende Tests, die Liste wird kontinuierlich aktualisiert:

Aktenzeichen der Sonderzulassung des BfArM	Hersteller	Antragsteller	Testname	BfArM-AT-Nummer*
5640-S-168/21	Hangzhou Clongene Biotech Co., Ltd.	Hangzhou Clongene Biotech Co., Ltd.	COVID-19 Antigen Rapid Test	

Overview of CLUNGENE® COVID-19 Antigen Rapid Test (*For Self-Testing*)

The self-testing product is transformed from the professional-testing product (**CLUNGENE® COVID-19 Antigen Rapid Test**) manufactured by Clongene. The BfArM has granted the first special approvals according to §11 paragraph 1 of the German Medical Devices Act (MPG) of antigen tests for self-administration by laypersons (self-tests) for the detection of SARS-CoV-2. Our self-testing product can be sold and used in Germany.

Test Procedure

Take one-test kit as an example.

CLUNGENE® COVID-19 Antigen Rapid Test (For Self-Testing)

Test Preparation



[What is included in the test kit]




Test Cassette (1X)



Swab (1X)



Extraction Reagent (1X)

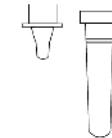

Tube (1X)

Instructions for Use is also included.

Sample Collection




Swab Left Nostril



Swab Right Nostril

■ Sample Preparation

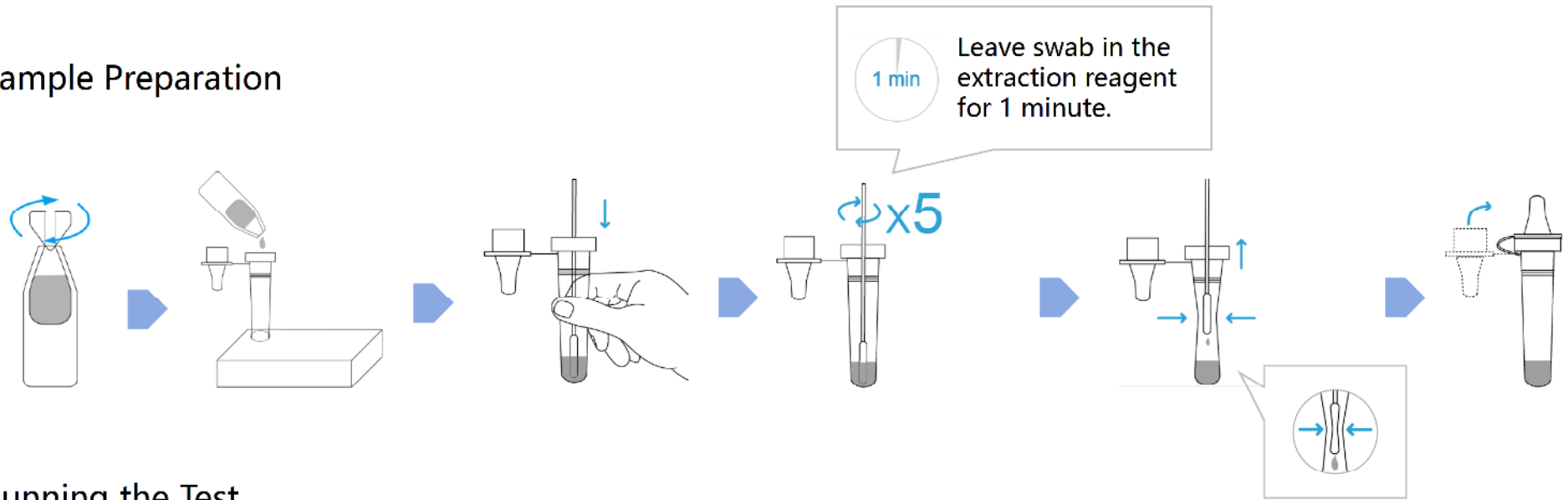

■ Running the Test



BACK

02

For Professional

CLUNGENE® COVID-19 Antigen Rapid Test



National List

CLUNGENE® COVID-19 Antigen Rapid Test (For Professional)

Validated by Paul-Ehrlich-Institut (PEI) in Germany.

Validated by Nationalen Referenzzentrum für neu auftretende Virusinfektionen (NAVI) in Switzerland, with comments as "Diese Ergebnisse liegen deutlich über den Empfehlungen der WHO für Ag-Schnelltests" by BAG.

BfArM

Listed by BfArM.

PEI

EU

Listed in the EU common list of COVID-19 rapid antigen tests on 17 February 2021.

NAVI

Performance Characteristics

CLUNGENE® COVID-19 Antigen Rapid Test (For Professional)

The self-testing product is transformed from the professional-testing product (**CLUNGENE® COVID-19 Antigen Rapid Test**) manufactured by Clongene.

■ Clinical Performance

617 nasal swabs were collected from individual symptomatic patients (within 7 days of onset) and asymptomatic patients who were suspected of COVID-19. The swabs were detected by COVID-19 Antigen Rapid Test of Clongene and the RT-PCR. Summary data as below:

COVID-19 Antigen		RT-PCR (Ct value ≤ 33)		Total
		Positive	Negative	
CLUNGENE®	Positive	132	3	135
	Negative	4	462	466
Total		136	465	601

PPA (Ct ≤ 33): 97.1% (132/136), (95%CI: 92.7% ~ 98.9%)
 NPA: 99.4% (462/465), (95%CI: 98.1% ~ 99.8%)

COVID-19 Antigen		RT-PCR (Ct value ≤ 37)		Total
		Positive	Negative	
CLUNGENE®	Positive	139	3	142
	Negative	13	462	475
Total		152	465	617

PPA (Ct ≤ 37): 91.4% (139/152), (95%CI: 85.9% ~ 94.9%)
 NPA: 99.4% (462/465), (95%CI: 98.1% ~ 99.8%)

PPA - Positive Percent Agreement (Sensitivity)
 NPA - Negative Percent Agreement (Specificity)